



## NOBASORB®

REF 840014

### Product Description and Purpose

The non-sterile, absorbent wound pad with a size of 15cm x 20cm consists of a soft nonwoven which wraps an absorbent core of soft, bleached fluff pulp and a cellulose layer for the distribution of secretions. On its back, the pad is provided with a nonwoven layer that is impenetrable to liquids. The absorbent wound pad is used for the treatment and cleansing of heavily exuding, infected or chronic wounds. It may also be used in the case of weeping wound healing disorders, for wound padding, and as secondary dressing for heavily exuding wounds. NOBASORB® is for single use only and must be sterilized with a validated procedure (moist heat at 121°C/134°C, 2 to 3 bar, in accordance with DIN EN ISO 17665 or ethylene oxide in accordance with DIN EN ISO 11135).

### Contra-Indications

All medium to lightly exuding wounds, in particular during granulation and epithelization stage, as well as deep and undermined wounds, should not be treated with absorbent wound pads.

The product should not be used in the case of a known allergy to one of the ingredients listed below.

### Composition

Polypropylene, cellulose

### Normative and Legal Requirements

NOBASORB® Absorbent Pad is a medical product according to the MPG, the directive 93/42/EEC and the regulation MDR (EU) 2017/745 and is classified as class I, rule 4.

The product does not contain dangerous toxic substances according to REACH. It has CE marking and DIN EN ISO 15223-1 labels on all its packaging.

### Packaging

Primary packaging: paper bag

Secondary packaging: carton made of cellulose

### Storage

To be stored in a dry and dust-free environment

### The product bears following labels and symbols:



Sterilize before use